


FINGER WRAP

NIPPON SIGMAX CO.,LTD.
33rd Floor Shinjuku Oak Tower
6-8-1 Nishi-shinjuku, Shinjuku-ku,
Tokyo 163-6033 JAPAN
Phone: +81 3 5326 3229


How to apply - Illustrations

[FINGER WRAP Single]

1

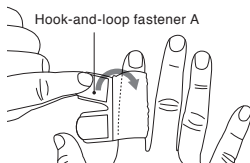


2

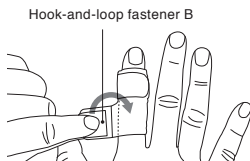

[FINGER WRAP Double]

3

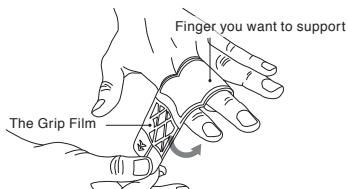

ZAMST FINGER WRAP Single/Double INSTRUCTIONS FOR USE

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use, and store it away for future reference.

Usage:

Finger (except for thumb) support

Read this first:

This manual applies to both FINGER WRAP Single and FINGER WRAP Double.

This product can be used for the left or right side.

Support may not be provided by this product under certain usage conditions.

This product cannot be used to treat finger disease, disorder or injury.

Be sure the product size is correct before use. Refer to the size chart.

When using this product:

Due to the characteristics of materials used in this product, it may change colors in some storage conditions.

When storing this product, do not hang excessive weight on its hook-and-loop fastener or raised fabric.

Doing so may weaken the hook-and-loop fastener capability.

Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

WARNING: Using the product incorrectly may result in serious injury.

CAUTION: Using the product incorrectly may result in minor injury, or cause damage to property other than this product.

WARNING

To prevent abnormalities or worsening of symptoms, those who suffer from the following conditions should refrain from using the product.

- Those predisposed to allergies or with sensitive skin.
- Those who have a fracture, cut, numbness, swelling, eczema, rash, or other abnormality in the area to be worn.

Be sure to follow the rules below and use the product correctly. If not used correctly, it could cause circulatory issues or injury.

- Do not use this product for any parts other than fingers.
- Be sure to use a product of the correct size. Refer to the size chart.
- Do not use while sleeping.
- When wearing the product for a long period of time, readjust as needed. Constant pressure for a long period of time may cause skin or circulatory issues.
- Use inside/outside and front/back correctly.
- Follow the steps for wearing this product to use it correctly.
- Do not fasten the main body of the product too tightly.
- If the product loosens or comes off during use, be sure to refasten it correctly by following steps from the beginning.
- If this product shows wear or damage or the hook-and-loop fastener no longer works well, do not use it.
- Do not attempt to process, modify, or repair this product.

If any of the following abnormalities occur when using this product, immediately stop using the product, and consult with a physician. Continued use may worsen symptoms.

- If it does not fit comfortably, etc.
- If an injury, fracture, cut, pain, numbness, swelling, eczema, rash, etc. occurs in the area the product is worn.

CAUTION

Hard materials are used for the hook-and-loop fastener. In some rare cases, fingers or skin may be damaged, so please use the product carefully.

Wear the product directly on the skin. It must not be worn over clothing, etc.

Sweat and friction may cause this product's color to fade, or move to other materials.

Make sure the hook-and-loop fastener does not attach to clothing. Doing so may cause it to become frayed or run.

Care and cleaning:**CAUTION**

Do not wash together with items faded in color. Doing so may cause the color to fade or move to other materials.

When washing together with clothing etc., be sure to use a laundry net. If the hook-and-loop fastener attaches to clothing, it may cause them to become frayed or run.

This product can be washed in a washing machine. After washing, hang it in the shade to dry.

When washing this product, attach the hook-and-loop fastener to the main body of the product.

Attempting to wash this product without closing the hook-and-loop fastener may weaken its fastening capability.

Do not use an iron or chlorine bleach. However, oxygen bleach can be used.

Do not use a dryer. Doing so may damage the product.

This product cannot be dry cleaned. Wet cleaning can be done, but avoid tumble drying.

How to apply:

Before attaching this product, carefully read through the [Precautions for use - Be sure to read this]. Then follow the steps provided for proper attachment.

[FINGER WRAP Single]

The attachment steps use the right (index) finger for reference. The same principles apply to the other fingers and to the left fingers as well.

1. Put the finger you want to support through the product so that the side with non-slip film is on the palm side of your hand (The side with the logo mark should come to the base of your finger). Fold back the excess part of the product (the dotted line in the figure) and fasten hook-and-loop fasteners A and B to the product (Illustrations 1 and 2).
2. If the tip of the hook-and-loop fastener touches the finger next to the supported finger, it is necessary to unfasten the hook-and-loop fasteners and adjust the position by turning the product, so that the hook-and-loop fasteners do not touch the adjacent finger.

[FINGER WRAP Double]

The attachment steps use the left (middle) finger for reference. The same principles apply to the other fingers and to the right fingers as well.

1. Put the finger you want to support through the loop part of the product. When doing so, put your finger through the loop so that the strap part of the product is on the side adjacent to the finger to which the strap will be fastened. The grip film should face up on the palm (Illustration 3).

* When you want to support your little finger, put your ring finger through the loop part of the product.

2. Wrap the strap part of the product around both fingers and fasten the hook-and-loop fasteners to secure the product.

* The logo position may be reversed, depending on which fingers are being strapped together.

Hold and pull each of the hook-and-loop fastener to fasten them on the main body of the product.

Hold down the hook-and-loop fasteners to fasten them firmly.

Materials:

Fiber: Nylon, Elastane

Non-fiber: Polyurethane

* Note: When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.

Disclaimer:

* Product specifications, external view, etc. may be changed without prior notice for improvement purposes.

* All possible measures were taken to ensure product quality, but if a defect, etc. is found, please contact us.