




NIPPON SIGMAX CO.,LTD.
33rd Floor Shirijuku Oak Tower
6-8-1 Nishi-shirijuku, Shirijuku-ku,
Tokyo 163-6033 JAPAN
Phone: +81 3 5326 3229



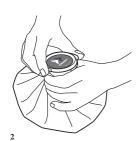



3





ZAMST ICE BAG INSTRUCTIONS FOR USE

Please read through the following instructions completely before using the product.

Intended use: Container for ice and cold water

Do not use this product in the following cases:

- If you have abnormal conditions such as cuts or rash in the area where this product is applied.

Safety precautions:

- If you experience pain, skin irritation, discomfort or any other abnormal reaction while using this product, discontinue use and consult with a physician.
- If your skin is sensitive or you have allergies, you may develop a rash or allergic reactions while using the product. If such symptoms occur, discontinue use and consult with a physician.
- Do not apply an ice-filled ice bag to the body for an extended period. It may cause frostbite.
- If you have nerve damage caused by conditions such as diabetes, limb paralysis or another disorder at the location to be iced, consult a medical professional before use. If an adverse reaction were to occur, you may not be able to feel it.
- If you use bandages to hold the ice bag in place, do not wrap the bandages too tightly.

To prevent product damage:

- If you use the product while it is wet, it may become discolored and its dye may bleed onto other clothing under or near the product.
- Condensation may appear on the surface of the bag, which may get clothing or skin wet.
- Avoid using ice with sharp edges. It may damage the inside of the bag, causing water to spill.
- Avoid storing the ice bag in areas with high temperatures, high humidity levels and direct sunlight exposure. These conditions may compromise the product's quality.

How to apply:

- 1. Fill the ice bag with ice until it is approximately 3/4 full. If you add some water with the ice, the bag will follow the shape of the body better. (Illustration 1)
- 2. Squeeze as much air out of the bag as possible and screw the cap on tightly. (Illustration 2)
- 3. Apply the bag to body part. You may use bandages to create compression and hold the bag in place. (Illustration 3)

How to wash:

- If the ice bag is soiled, wipe it with a damp cloth or hand wash with mild soap.
- Do not dry clean or iron. Do not use products containing chlorine, including bleach. These may damage the product.
- After using or washing, empty all water from the bag and let it air dry before putting it away.

Composition: Polyester, polyurethane, ABS foam, silicon and aluminum