

KT

ZAMST

Kinesiology tape for muscle support INSTRUCTIONS FOR USE

Intended use

- Muscle support
- Joint and muscle compression and stabilization

Descriptive labeling

Material		
Material: cotton, polyurethane	Adhesive material: acrylic adhesive	Release paper: paper

After use, please dispose of this product according to the rule of your local government.

Product specification

Specification (WidthxLength)	Product Number	Quantity per box
38mm × 5.0m	308701	2 rolls
50mm × 5.0m	308702	1 roll
75mm × 5.0m	308703	1 roll



The length is actual length.

In use of the product

- Keep this product out of reach of children avoiding heat, moisture and direct sunlight.
- To remove cleanly after use, peel off slowly from the edge of the tape while holding on the other side.

Safety precautions - please be sure to read -

In this instruction, safety precautions are classified depending on the level of risk as follows:

-  **WARNING: If the product is used improperly, serious injuries may result.**
-  **CAUTION: If the product is used improperly, minor personal injuries or damages to the equipment may result.**



To avoid abnormal conditions or aggravation of symptoms, do not use this product in the following cases:

- If your skin is sensitive or you have allergies.
- If you have abnormal conditions such as fractures, cuts, numbness, swelling or rash in the area where the product is used.

Use properly according to the following instructions. Improper use may cause poor blood circulation or injuries.

- Do not apply this product to the skin for a long time and not peel off it quickly. It may cause skin irritation.
- Do not strain or tighten strongly this product.

If the following abnormal conditions occur while using this product, discontinue use and consult with a physician. Continuing use under such conditions, the symptoms may worsen.

- If you have abnormal conditions such as injury, fractures, cuts, pains, numbness, swelling, rash, redness, itchy and skin irritation in the area where product is used.

NIPPON SIGMAX CO.,LTD.
33rd Floor Shinjuku Oak Tower
6-8-1 Nishi-shinjuku, Shinjuku-ku,
Tokyo 163-6033 JAPAN
Phone: +81 3 5326 3229